




ZIRCONIA
Blazir

Blazing the way
to simple sintering

3 Start a program

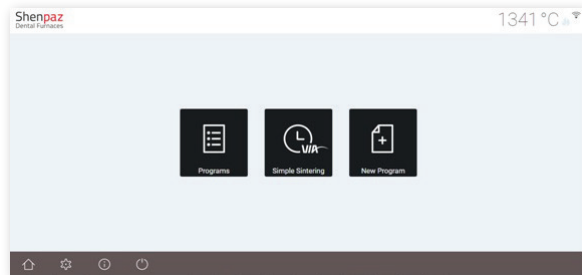

There are 3 possibilities:
From the opening screen,
select from preset PROGRAMS,
NEW PROGRAM or
VIA (optional feature).
To add a new program,
please refer to the enclosed
user manual.

5 HaTaasia Street
Ramat Gabriel Ind. Park,
Migdal HaEmek 23101, Israel
Tel: +972 (0)3 562 0428
Fax: +972 (0)4 6042040
www.shenpaz.com

Shenpaz
Dental Furnaces

ZIRCONIA
Blazir

Blazing the way
to simple sintering



Get Started Guide

Shenpaz
Dental Furnaces




Blazing the way
to simple sintering

Thank you for purchasing the "Blazir" sintering furnace



Assure Blazir is disconnected from power supply.

1 Installing the heating elements

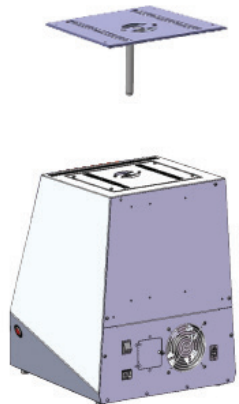

Step 1

Take off the top cover with exhaust airpipe by unscrewing the 4 screws located in each corner of the top




Step 2

From the accessory bag take the connection metal strips, tools, and extra screws



Step 3

Carefully unpack each heating element from their box and place into furnace one by one. NOTE: heating elements are very fragile.

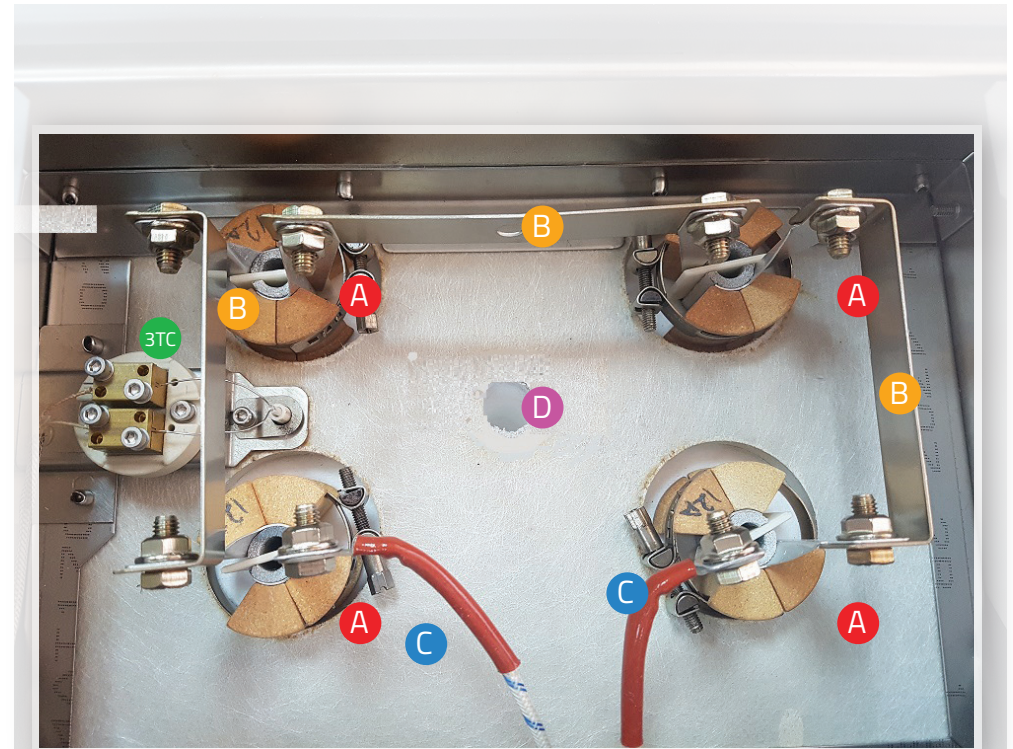

A Insert each of the 4 heating elements into their placement on the top of the muffle, paying attention that the bottom of the HE is also correctly aligned in the base of the muffle.

B THIGHLY connect the 3 metallic strips with provided hexa screws as per image above. NOTE exact position of strips.

3TC Check that the TC is not in contact with the metallic strips

C Connect each power cable with screws to the respective heating element (see above)

D Close top of furnace by screwing the cover back on. ASSURE that the edge of exhaust pipe is in hole.

2 Connect furnace to main current and turn on the furnace. BLAZIR is sinter ready!